

STAT PORTABLE PROCEDURE

Website: <https://oasis.umedex.com>

username: **imam**


password: **imam#**

Note:



- ❖ Umedex and stat portable websites are same. Just login credentials are different and should open in internet explorer only.
- ❖ Before opening any stat portable case, first check for prior.
- ❖ Prior will not get opened on a same system.
- ❖ Prior should open on a separate system only.
- ❖ Note: Prior should open on a separate system. Before opening any stat portable case, we have to check for prior first. If prior is there, we should open current case on one system and prior on another system.
- ❖

1. First open internet explorer.

2. Open the stat portable website in internet explorer.

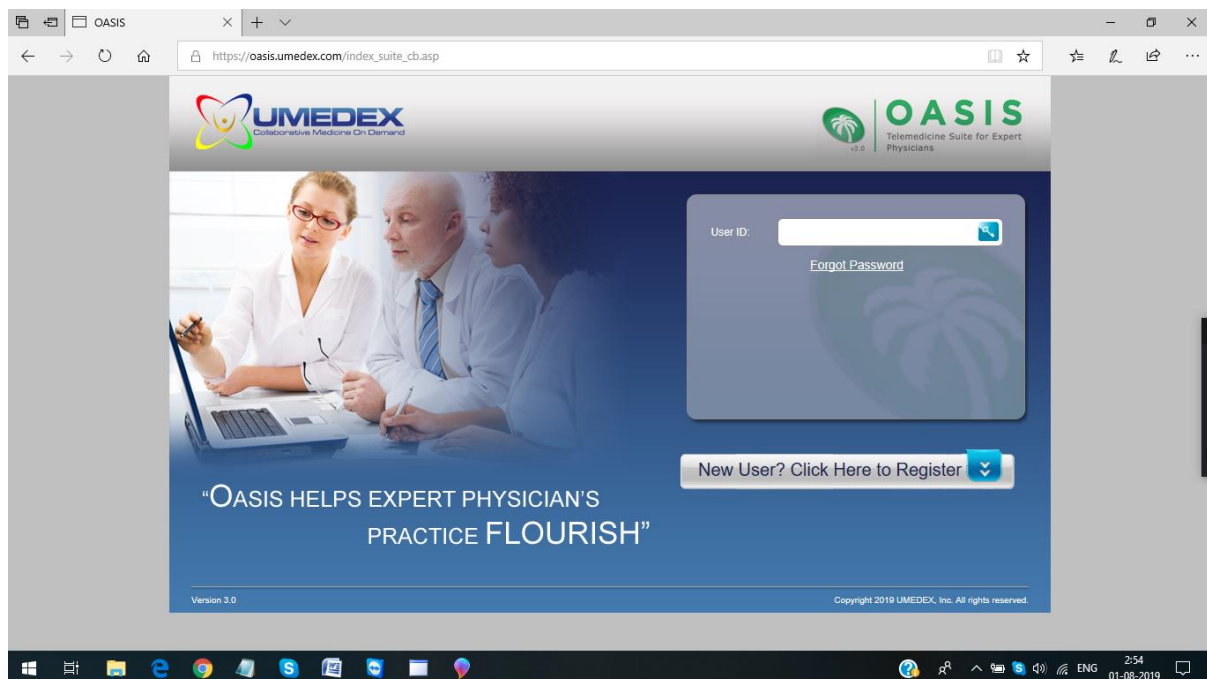


Username: **imam** – NEXT - Click on the **Key symbol**.

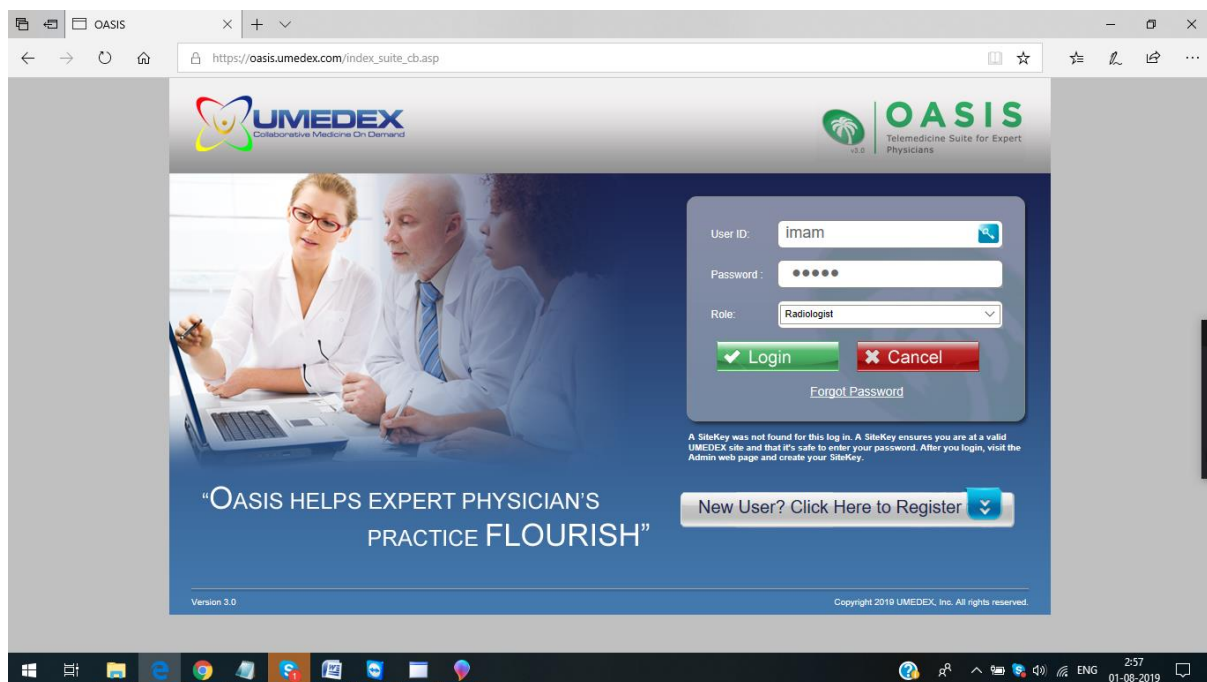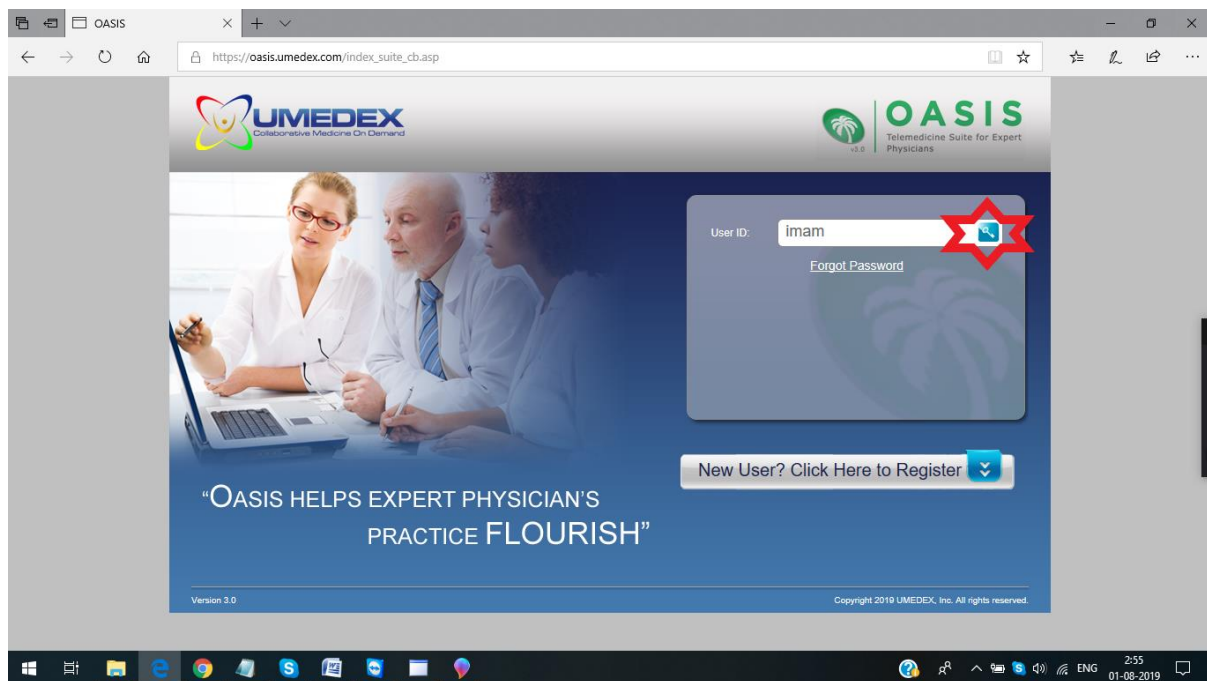

Password: **imam#**


NEXT - Click on **LOGIN**.

Close the POP-up – **User profile Webpage Dialogue**

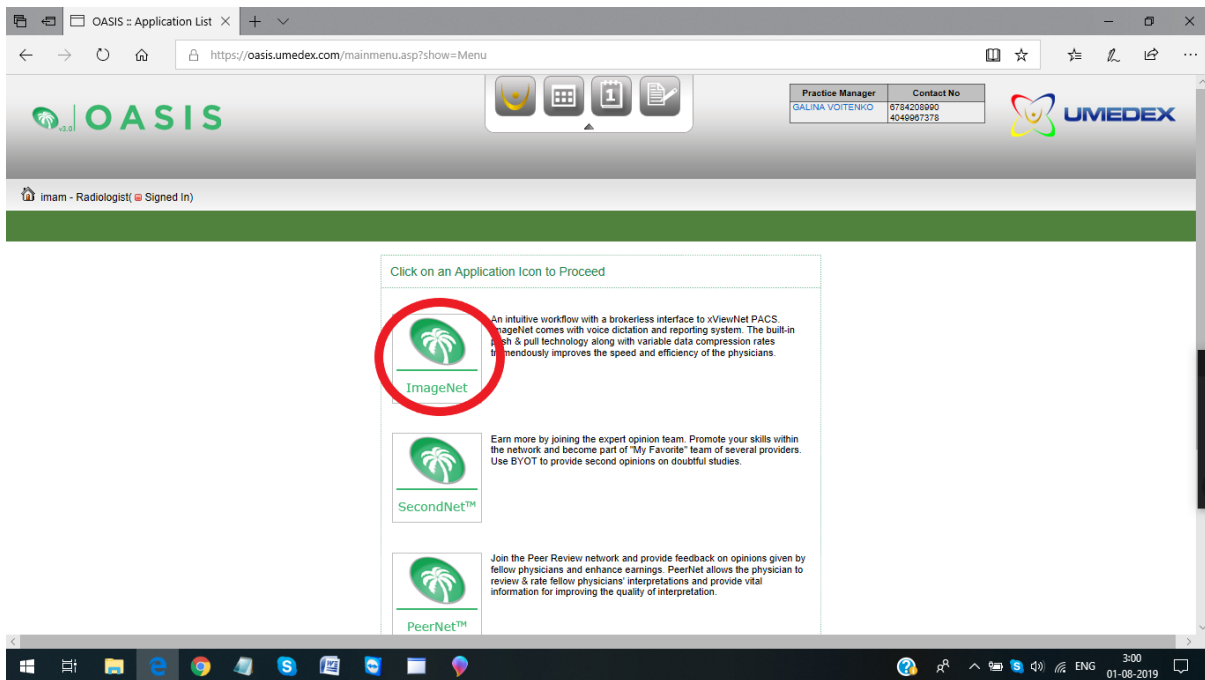

Next – Click on “**Image Net**” (An intuitive workflow with a brokerless interface to xViewNet PACS.)

This is **stat portable x-ray---US** (ultrasound) worklist

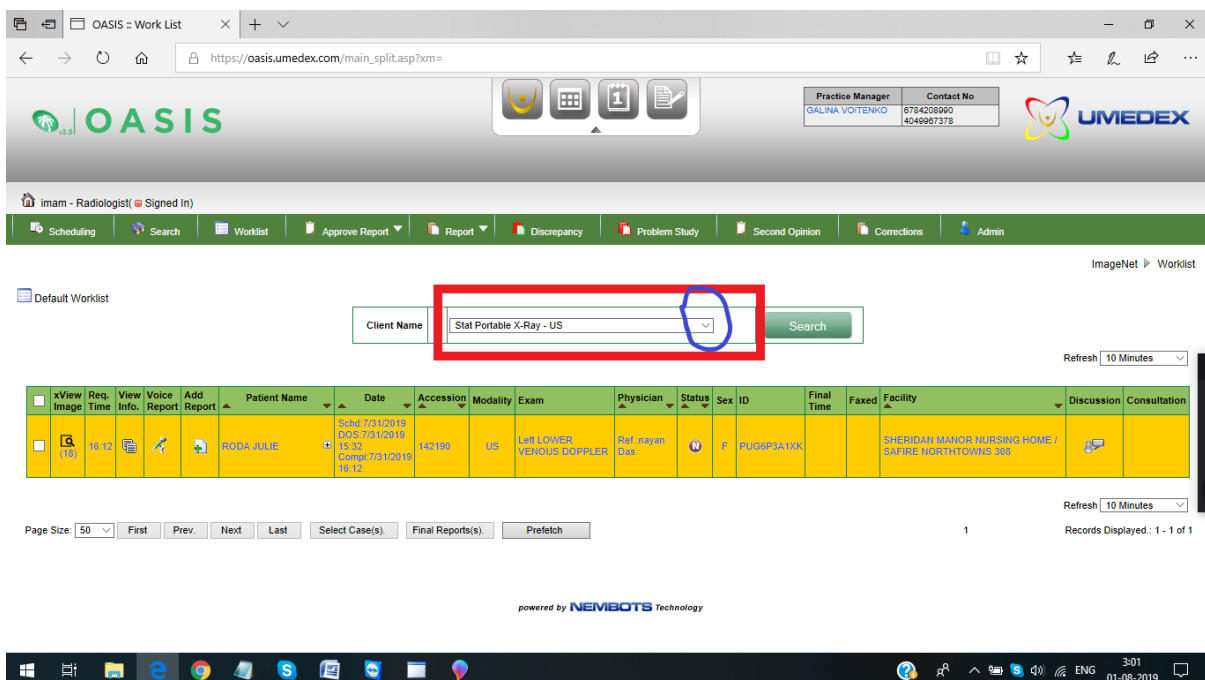

Click on **down arrow button** to get X-ray worklist

Select the **stat portable X-ray---X-ray**

The screenshot shows the OASIS Work List application interface. The search bar at the top contains the text 'Stat Portable X-Ray - X-Ray'. Below the search bar, a table displays the search results. The table has columns for xView, Req. Time, View Info, Voice Report, Add Report, Patient Name, Date, Accession, Modality, Exam, Physician, Status, Sex, ID, Final Time, Faxed, Facility, Discussion, and Consultation. The first row of data shows a patient named 'RODA JULIE' with a date of '7/31/2019' and an accession number of '142190'. The modality is 'US' and the exam is 'Left LOWER VENOUS DOPPLER'. The physician is 'Ref. mayan Das'. The status is 'U' and the sex is 'F'. The ID is 'FUGBP3K1XK'. The final time is '18:12'. The facility is 'SHERIDAN MANOR NURSING HOME / SAFIRE NORTH TOWNS 308'. The discussion and consultation columns are empty.

xView	Req. Time	View Info	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
	18:12				RODA JULIE	7/31/2019 18:12	142190	US	Left LOWER VENOUS DOPPLER	Ref. mayan Das	U	F	FUGBP3K1XK	18:12		SHERIDAN MANOR NURSING HOME / SAFIRE NORTH TOWNS 308		

Stat portable X-ray worklist

The screenshot shows the OASIS Work List application interface. The search bar at the top contains the text 'Stat Portable X-Ray - X-Ray'. Below the search bar, a table displays the search results. The table has columns for xView, Req. Time, View Info, Voice Report, Add Report, Patient Name, Date, Accession, Modality, Exam, Physician, Status, Sex, ID, Final Time, Faxed, Facility, Discussion, and Consultation. The first row of data shows a patient named 'PARDIEU GENIAL' with a date of '7/31/2019' and an accession number of '142190'. The modality is 'DX' and the exam is 'CHEST'. The physician is 'Ref. Shrawen Kumar Madaga'. The status is 'U' and the sex is 'M'. The ID is 'DESCAGJFMPVY'. The final time is '18:30'. The facility is 'Bainbridge Nursing Unit 2, 201A'. The discussion and consultation columns are empty.

xView	Req. Time	View Info	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
	18:30				PARDIEU GENIAL	7/31/2019 18:30	142190	DX	CHEST	Ref. Shrawen Kumar Madaga	U	M	DESCAGJFMPVY	18:30		Bainbridge Nursing Unit 2, 201A		

To open a **case**, click on the **magnifier (view image column)** icon.

To open the **Report**/document–Click on **Add Report/Plus Symbol +**.

Once the case is opened, follow the 7 steps.

Client Name: Stat Portable X-Ray - X-Ray Search


xView Image	Req. Time	View Info	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
[G] (1)	17:46	[G] (1)	[G] (1)	[+]	PI NI DONALD	Schd: 7/31/2019 DOS: 7/31/2019 17:12 Comp: 7/31/2019 17:46	142161	DX	CHEST	Ref: Joseph Tu	[U]	M	UJWA60XXYZ			Townhouse Rehab 4378	[G] (1)	
[G] (1)	17:46	[G] (1)	[G] (1)	[+]	ROGERS BEVERLY	Schd: 7/31/2019 DOS: 7/31/2019 17:06 Comp: 7/31/2019 17:46	142154	DX	CHEST	Ref: Joseph Tu	[U]	F	GABR20PHGI			Townhouse Rehab 1198	[G] (1)	
[G] (1)	18:30	[G] (1)	[G] (1)	[+]	HARDIEU GENIAL	Schd: 7/31/2019 DOS: 7/31/2019 18:00 Comp: 7/31/2019 18:30	142188	DX	CHEST	Ref: Shrawan Kumar Mallapu	[U]	M	DSCA6JFMPW			Bethbridge Nursing Unit 2,2014	[G] (1)	

After opening the report – Follow the 7 steps.

Please check for **History**, number of views, **DOB**.


(Comparison should open on a separate system only. Before opening a case, check for prior and do accordingly).

For History see “**Study Information –Indication**”.

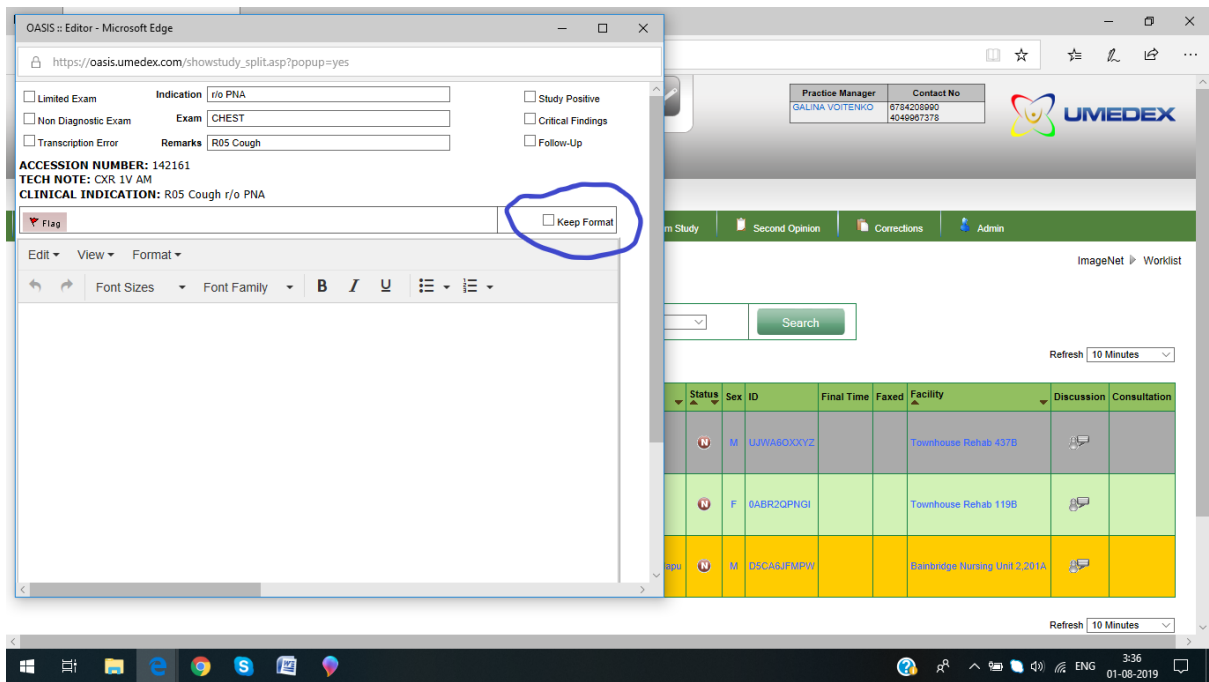

Click on that “Keep format” option on right side, which keeps the report in format. If we do not click on that option, report will not be in a format.


Then paste the macro in report.

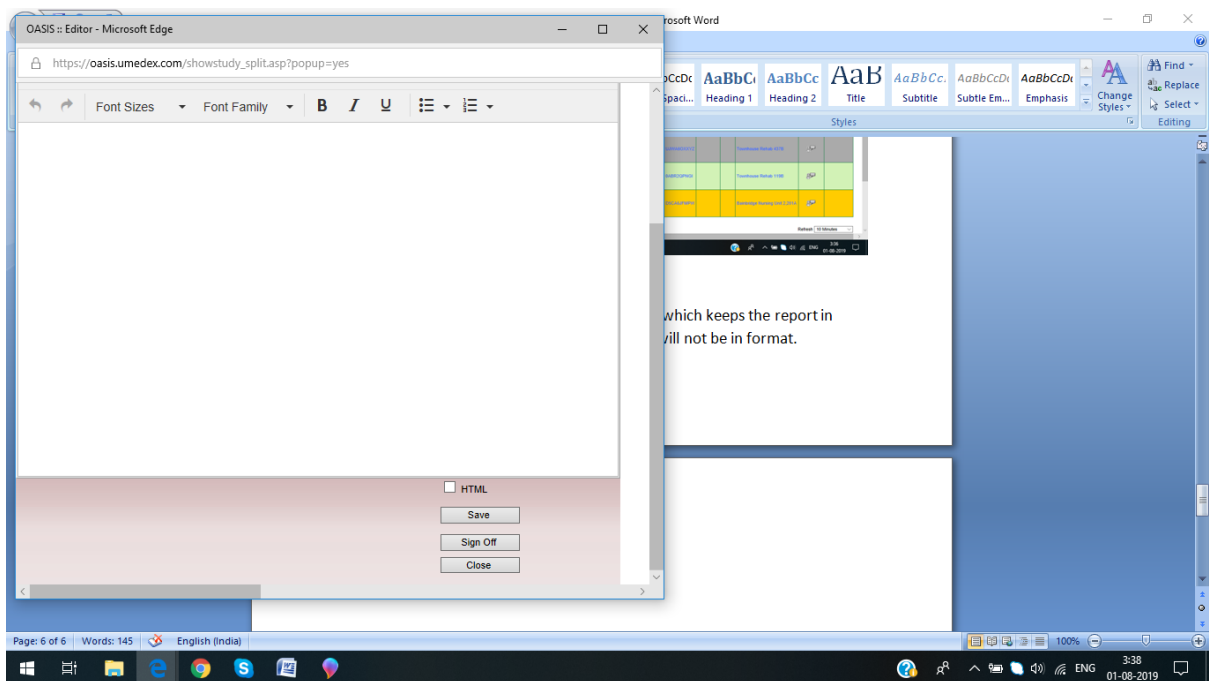

Once the report is completed - Click on sign off option.

Next – Message from WebRadiology “Have you answered the requested the radiological evaluation ?

Click on OK option.

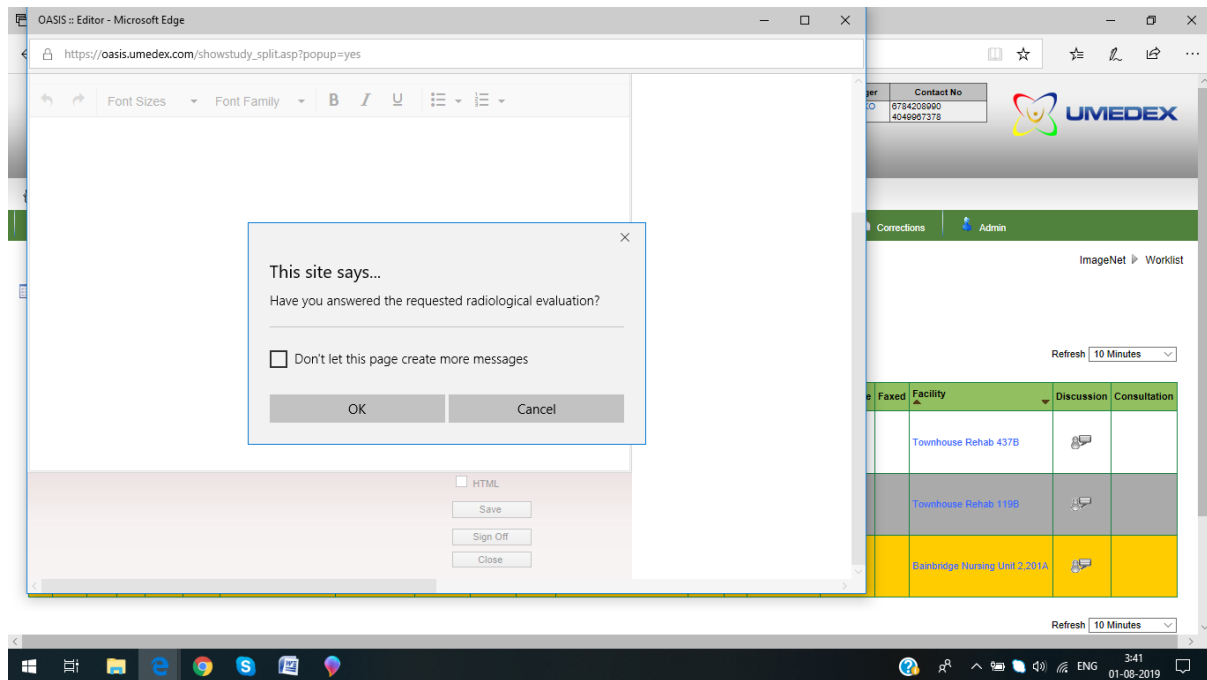


Click on “OK”

To check for **PRIOR**: click on the small “+button”

The screenshot shows the OASIS Worklist application interface. At the top, there's a navigation bar with icons for Home, Calendar, and a document. Below that, a menu bar includes options like Scheduling, Search, Worklist, Approve Report, Report, Discrepancy, Problem Study, Second Opinion, Corrections, and Admin. The main area is titled 'Default Worklist' and features a search bar with 'Stat Portable X-Ray - X-Ray' entered. A table lists patients with columns for xView, Req. Time, View Info, Voice Report, Add Report, Patient Name, Date, Accession, Modality, Exam, Physician, Status, Sex, ID, Final Time, Faxed, Facility, Discussion, and Consultation. The patient 'PISANI DONALD' is highlighted, and a red box is drawn around the '+' button next to their name. The bottom of the screen shows a Windows taskbar with various application icons and system clock.

This screenshot shows the same OASIS Worklist application, but with a 'Patient History' pop-up window open. The pop-up window has a search bar and a table with columns for Patient Name, Date, Modality, Status, Exam, and Facility. The patient 'PISANI DONALD' is listed multiple times with different dates and accession numbers. A red box highlights the '+' button next to the patient name 'PISANI DONALD' in the main table, and another red box highlights the '+' button next to the patient name 'PISANI DONALD' in the pop-up window. The bottom of the screen shows a Windows taskbar with various application icons and system clock.

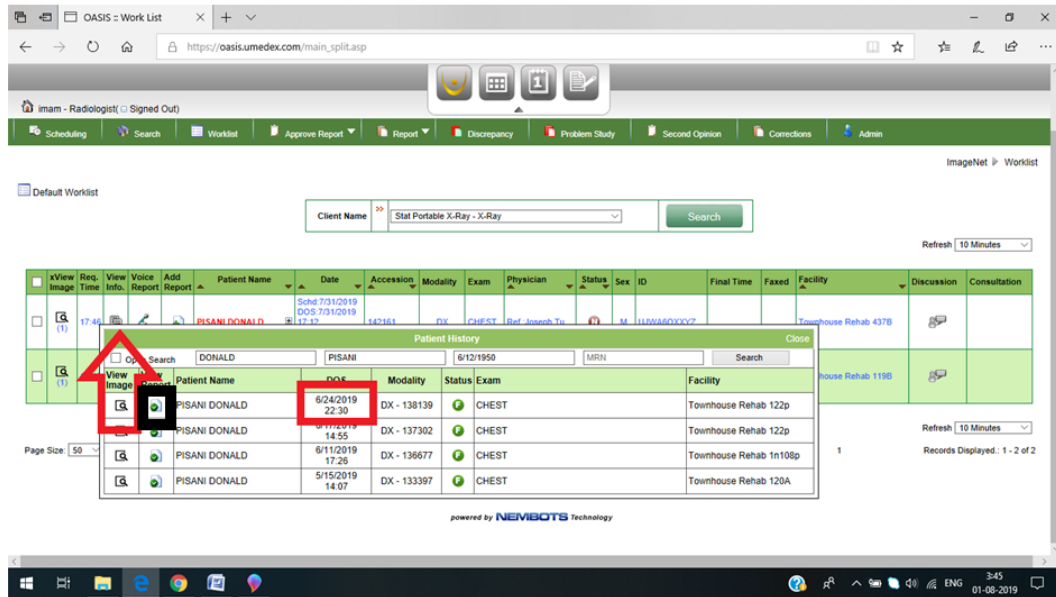

Click on the recent prior date:

View image (magnifier icon) to open prior images

View report (document icon) to open prior report.

Note: Prior should open on a separate system. Before opening any stat portable case, we have to check for prior first. If prior is there, we should open current case on one system and prior on another system.

OASIS - Work List

https://oasis.umedex.com/main_split.asp

imam - Radiologist (Signed Out)

Scheduling | Signed In | At Work | At Lunch | Personal Work | Signed Out

Worklist | Approve Report | Report | Discrepancy | Problem Study | Second Opinion | Corrections | Admin

ImageNet Worklist

Client Name: Stat Portable X-Ray - X-Ray Search

Refresh 10 Minutes

xView Image	Req. Time	View Info	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
	17:46				PISANI DONALD	Sched: 7/31/2019 17:12 DOS: 7/31/2019 17:46	142161	DX	CHEST	Ref. Joseph Tu		M	UJWA60XXYZ			Townhouse Rehab 437B		
	17:46				ROGERS BEVERLY	Sched: 7/31/2019 17:06 DOS: 7/31/2019 17:46	142154	DX	CHEST	Ref. Joseph Tu		F	0ABR2QPNGI			Townhouse Rehab 119B		

Page Size: 50 First Prev Next Last Select Case(s) Final Reports(s) Prefetch

1 Records Displayed: 1 - 2 of 2

Refresh 10 Minutes


powered by NEMEBOTS Technology

Windows taskbar: 3:49 01-06-2019


Once done with stat portable cases, **Signed out** from stat portable website.

Stat portable procedure screenshots


1.

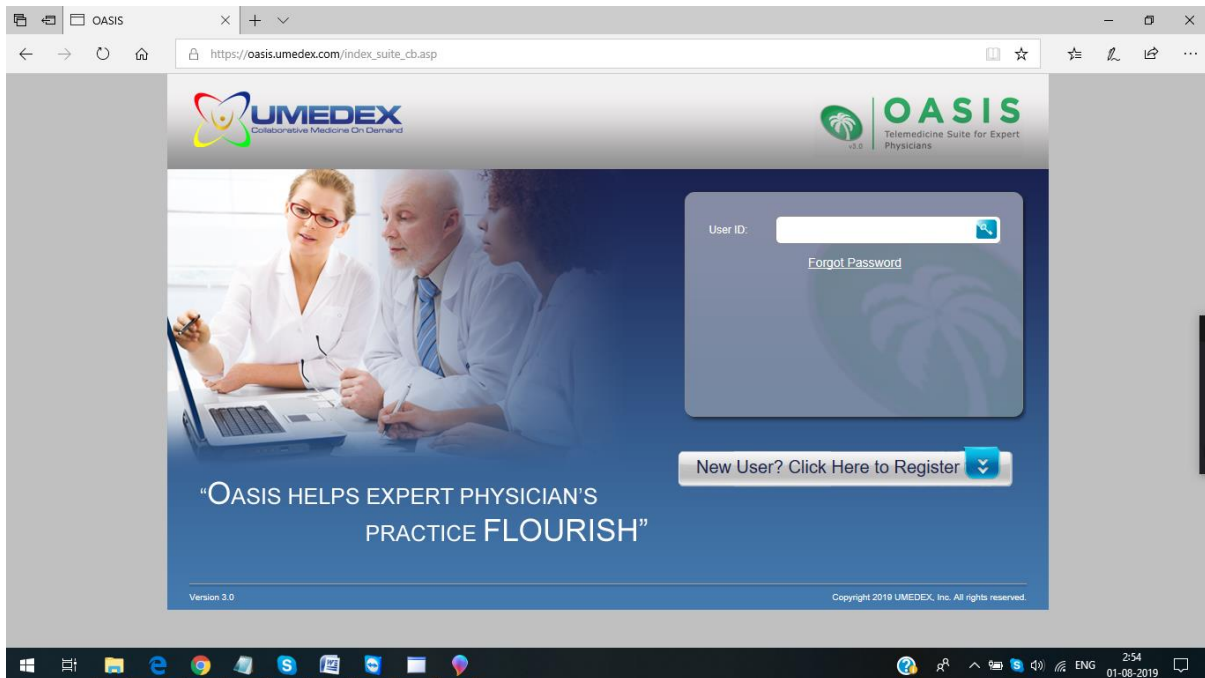


2.

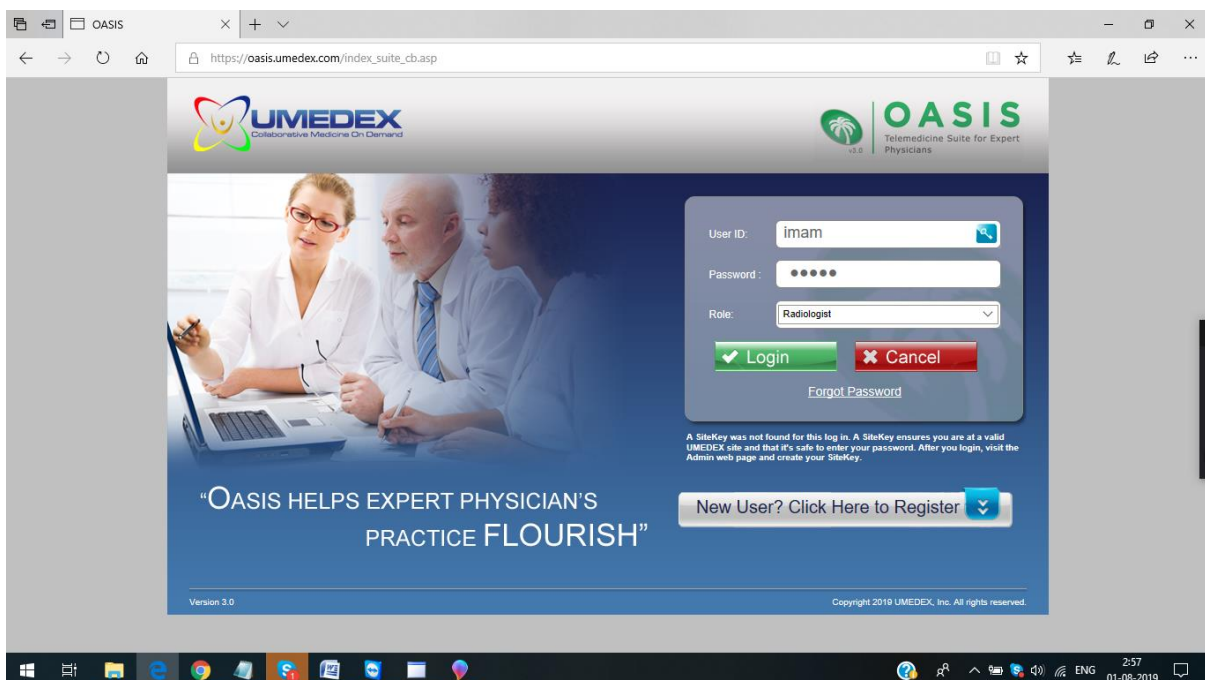

3.

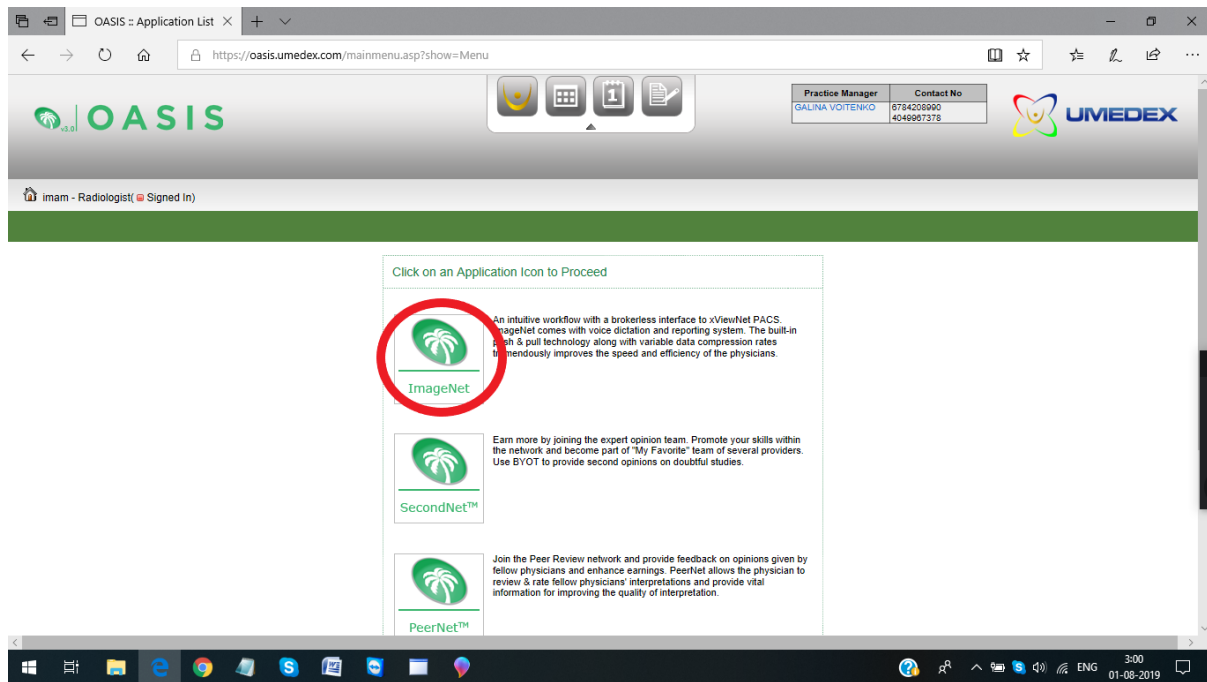

4.

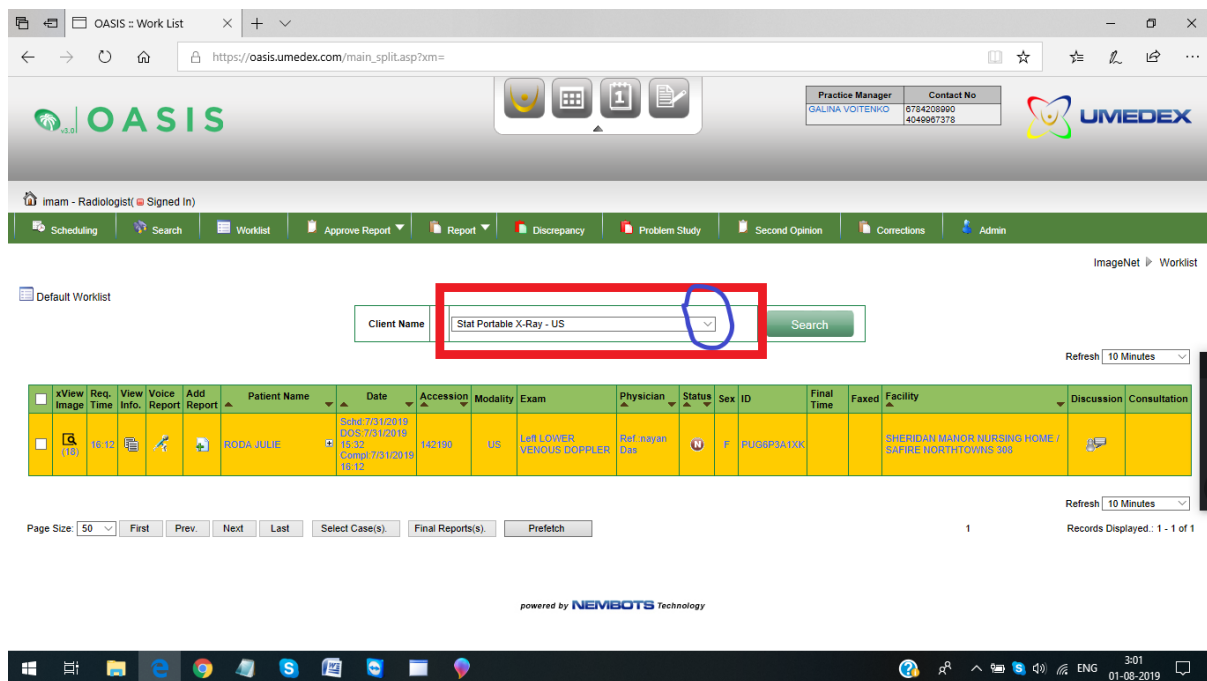

5.

The screenshot shows the OASIS Work List interface. The search bar is highlighted with a red box, showing the search criteria: "Stat Portable X-Ray - US" and "Stat Portable X-Ray - X-Ray". The search results table is displayed below, showing a single entry for a patient named RODA JULIE.

xView Image	Req. Time	View Info.	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
	16.12				RODA JULIE	Sched: 7/31/2019 15:32 DOS: 7/31/2019 15:32 Comp: 7/31/2019 16:12	142190	US	Left LOWER VENOUS CUPPLER	Ref: rajan Das	N	F	PUGBP3A1XN			SHERIDAN MANOR NURSING HOME / SAFIRE WORTHINGTONS 308		

Page Size: 50 | First | Prev. | Next | Last | Select Case(s) | Final Report(s) | Prefetch | 1 | Records Displayed: 1 - 1 of 1

powered by NEMEBOTS Technology

6.

The screenshot shows the OASIS Work List interface. The search bar is highlighted with a red box, showing the search criteria: "Stat Portable X-Ray - X-Ray". The search results table is displayed below, showing a single entry for a patient named PARDIEU GENIAL.

xView Image	Req. Time	View Info.	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
	16.30				PARDIEU GENIAL	Sched: 7/31/2019 15:05 DOS: 7/31/2019 15:05 Comp: 7/31/2019 16:30	142156	DX	CHEST	Ref: Shrawen Kumar Malhotra	N	M	BSCABJMPW			Bainbridge Nursing Unit 2.201A		

Page Size: 50 | First | Prev. | Next | Last | Select Case(s) | Final Report(s) | Prefetch | 1 | Records Displayed: 1 - 1 of 1

powered by NEMEBOTS Technology

7.

OASIS Work List

https://oasis.umedex.com/main_split.asp

Practice Manager: GALINA VOITENKO, Contact No: 6784208990, 4049987378

imam - Radiologist (Signed Out)

Scheduling, Search, Worklist, Approve Report, Report, Discrepancy, Problem Study, Second Opinion, Corrections, Admin

ImageNet Worklist

Default Worklist

Client Name: Stat Portable X-Ray - X-Ray

Refresh: 10 Minutes

View Image	View Info	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
				PI NI DONALD	Sched 7/31/2019 17:12 DOS: 7/31/2019 17:46 Compl 7/31/2019 17:46	142161	DX	CHEST	Ref: Joseph Tu		M	UJWA60XXYZ			Townhouse Rehab 437B		
				ROGERS BEVERLY	Sched 7/31/2019 17:06 DOS: 7/31/2019 17:46 Compl 7/31/2019 17:46	142154	DX	CHEST	Ref: Joseph Tu		F	6ABR20PNGI			Townhouse Rehab 119B		
				PARDIEU GENIAL	Sched 7/31/2019 15:09 DOS: 7/31/2019 16:39 Compl 7/31/2019 16:39	142108	DX	CHEST	Ref: Shrawan Kumar Mallapu		M	DSCABJFMPW			Bainbridge Nursing Unit 2.201A		

Refresh: 10 Minutes

8.

OASIS Editor - Microsoft Edge

https://oasis.umedex.com/showstudy_split.asp?popup=yes

☐ Limited Exam
☐ Non Diagnostic Exam
☐ Transcription Error

Indication: r/o PNA

Exam: CHEST

Remarks: R05 Cough

ACCESSION NUMBER: 142161

TECH NOTE: CXR 1V AM

CLINICAL INDICATION: R05 Cough r/o PNA

Flag:

Study Positive ☐

Critical Findings ☐

Follow-Up ☐

Keep Format ☐

Edit, View, Format

Font Sizes, Font Family, B, I, U,

Practice Manager: GALINA VOITENKO, Contact No: 6784208990, 4049987378

im Study, Second Opinion, Corrections, Admin


ImageNet Worklist

Refresh: 10 Minutes


Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
	M	UJWA60XXYZ			Townhouse Rehab 437B		
	F	6ABR20PNGI			Townhouse Rehab 119B		
	M	DSCABJFMPW			Bainbridge Nursing Unit 2.201A		

Refresh: 10 Minutes


9.

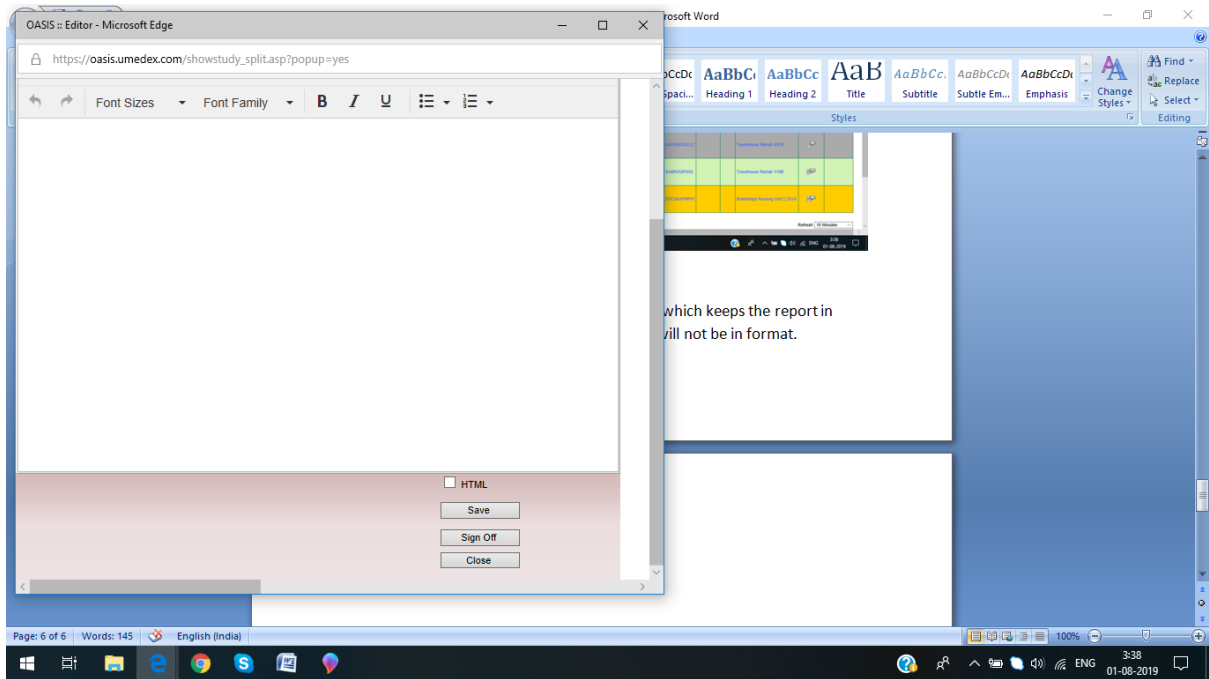

10.

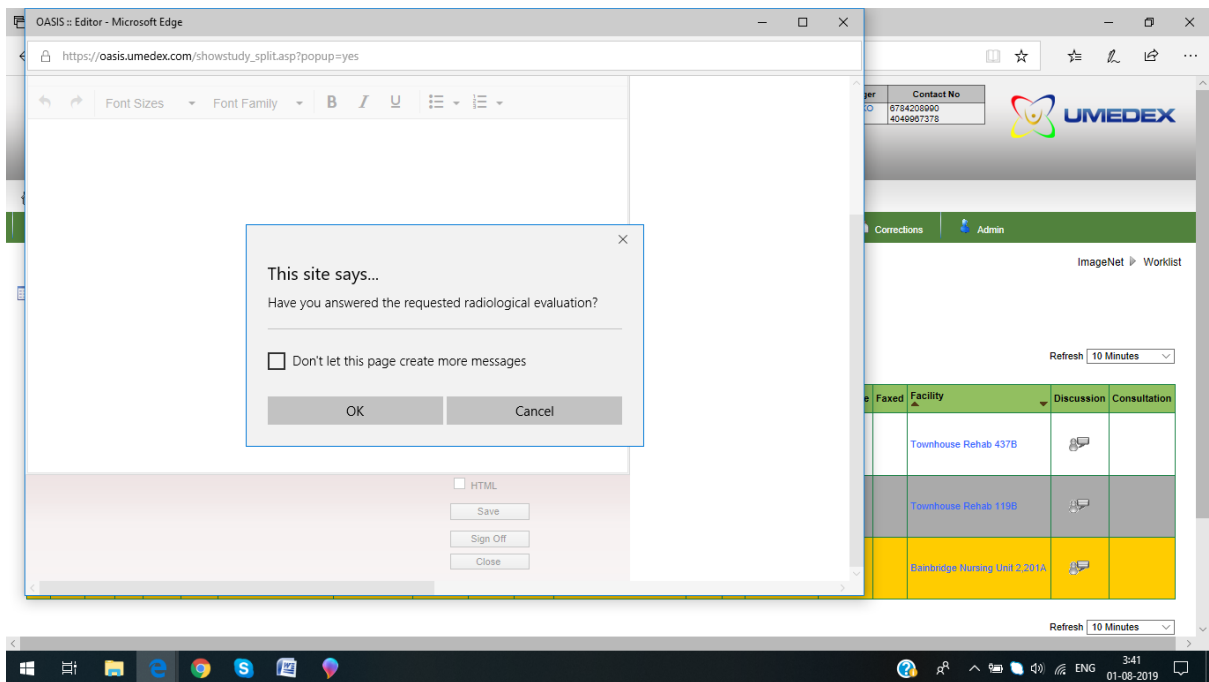

11. To check for prior:

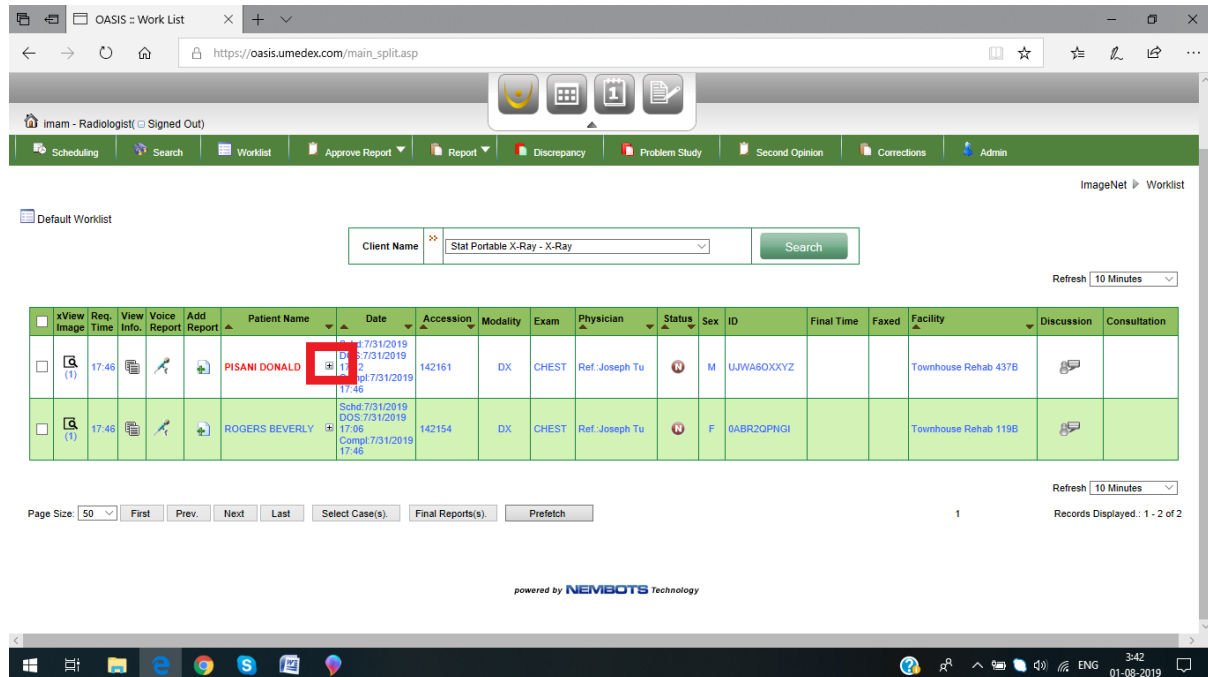


The screenshot shows the OASIS Work List application interface. The top navigation bar includes buttons for Scheduling, Search, Worklist, Approve Report, Report, Discrepancy, Problem Study, Second Opinion, Corrections, and Admin. The main content area displays a table of patients with columns for xView, Image, Req. Time, View Info, Voice Report, Add Report, Patient Name, Date, Accession, Modality, Exam, Physician, Status, Sex, ID, Final Time, Faxed, Facility, Discussion, and Consultation.

xView	Image	Req. Time	View Info	Voice Report	Add Report	Patient Name	Date	Accession	Modality	Exam	Physician	Status	Sex	ID	Final Time	Faxed	Facility	Discussion	Consultation
		17:46				PISANI DONALD	Schd 7/31/2019 17:06 DOS: 7/31/2019 17:06 Compl 7/31/2019 17:46	142161	CHEST	Ref. Joseph Tu		M	UJWA60XXYZ				Townhouse Rehab 437B		
		17:46				ROGERS BEVERLY	Schd 7/31/2019 17:06 DOS: 7/31/2019 17:06 Compl 7/31/2019 17:46	142154	CHEST	Ref. Joseph Tu		F	0ABR20PNGI				Townhouse Rehab 119B		

The 'View Image' icon for PISANI DONALD is highlighted with a red box. The application also includes a search bar for Client Name, a refresh button, and a footer indicating it is powered by NEMIBOTS Technology.

12.

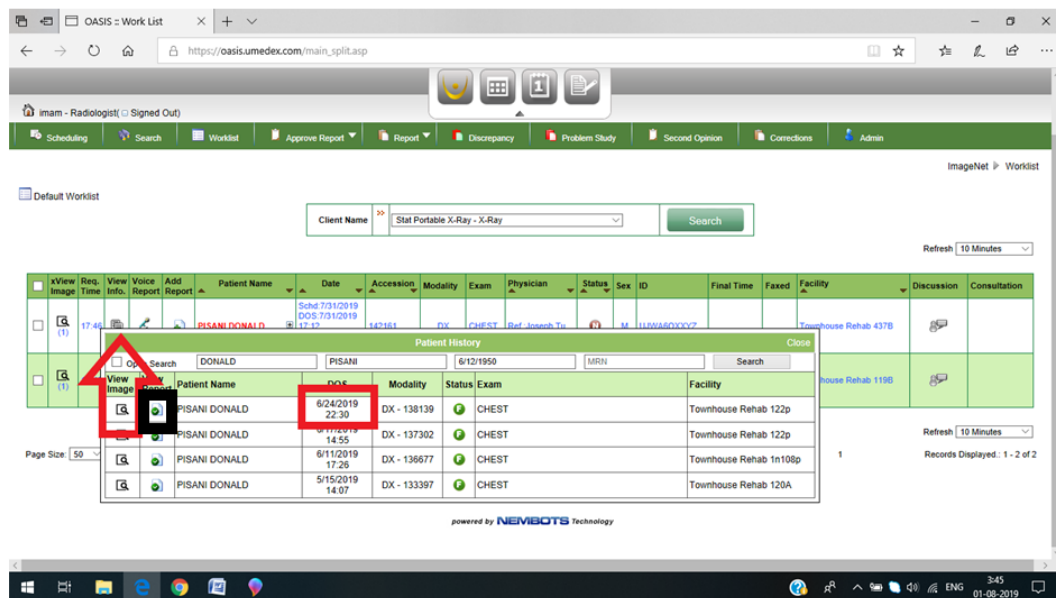

The screenshot shows the OASIS Work List application with a patient history pop-up for PISANI DONALD. The pop-up displays a table of patient history records with columns for Patient Name, Date, Accession, Modality, Status, Exam, and Facility.

Patient Name	Date	Accession	Modality	Status	Exam	Facility
PISANI DONALD	6/24/2019 22:30	DX - 138139	CHEST		Townhouse Rehab 122p	
PISANI DONALD	6/11/2019 14:55	DX - 137302	CHEST		Townhouse Rehab 122p	
PISANI DONALD	6/11/2019 17:26	DX - 136677	CHEST		Townhouse Rehab 1m108p	
PISANI DONALD	5/15/2019 14:07	DX - 133397	CHEST		Townhouse Rehab 120A	

The 'View Image' icon for the first record is highlighted with a red box. The application also includes a search bar for Patient Name, a refresh button, and a footer indicating it is powered by NEMIBOTS Technology.
